Reynard Premier detergent & disinfectant wipes

A single step solution to cleaning and disinfecting surfaces and equipment, proven to kill at least 99.999% of pathogens. With a superior cloth and unsurpassed equipment compatibility, germs don't stand a chance.

Specifications

- · Effective for use against COVID and SARS
- · Unparalleled equipment compatibility
- Superior cloth: up to 47% larger than competitors

Points to remember

- Use one wipe per piece of equipment or surface
- Wipe from clean to dirty
- Clean from top to bottom

Step 1: Peel back the sticky pack cover and take hold of a wipe and pull it out of the pack



Step 2: Open wipe to desired size and re-seal cover to retain moisture.



Step 3: Wipe surface in an 'S' shaped motion from top to bottom.



Step 4: Dispose of wipe. Do not flush or put into a macerator.



Please note: we recommend following hospital patient cleaning protocols, including the use of P.P.E. where appropriate.



SKU	WIPES PER PACK	PACKS PER CARTON
RHS216	50	100
RHS218	100	6

Reynard is reassurance.



New Zealand

PO Box 8470 Havelock North Sales@reynardhealth.com 0508 494 737

Australi

44 Hampden Road Artarmon, NSW, 2064 enquiries@reynardhealth.com.au 1300 667 199

Uŀ

White Horse Business Park, Standford in the Vale SN7 8NY www.reynardhealth.co.uk uk@reynardhealth.com +44 (0)845 263 7335

Internationa

PO Box 8470 Havelock North Sales@reynardhealth.com +64 (0)6 650 0709